

AV.ZI.TI. MAKE UP INSTANT



COLOUR CHART

Fluorescent Stains	9	IN-11 white	IN-12 yellow	IN-13 orange	IN-14 light brown	IN-15 olive	IN-16 dark brown	IN-17 red	IN-18 blue	IN-19 grey
	1	Glaze GL								
Make Up Instant Fluid	1	Fluid								

AV FIRING CHART

	Preheating	Drying Time	Raise of Temperature	V	Final Temperature	Holding Time	Appearance
Glaze Firing with Glaze Powder (Make Up Instant)	480°C	2 min.	45°C/min.	-	880°C	1 min.	Shining

ZI FIRING CHART

	Preheating	Drying Time	Raise of Temperature	V	Final Temperature	Holding Time	Appearance
Glaze Firing with Glaze Powder (Make Up Instant)	480°C	2 min.	45°C/min.	-	790°C	1 min.	Shining

TI FIRING CHART

	Preheating	Drying Time	Raise of Temperature	V	Final Temperature	Holding Time	Appearance
Glaze Firing with Glaze Powder (Make Up Instant)	480°C	2 min.	45°C/min.	-	780°C	1 min.	Shining

The firing parameters given above are guidelines, which always need to be adjusted to the furnace used for firing and the situation of the furnace.
The important thing is to obtain the right firing result.

AV.ZI.TI. MAKE UP INSTANT



INSTRUCTIONS FOR USE

RANGE OF APPLICATIONS

As Creation AV.ZI.TI. Make Up Instant stains are based on the basic shades, effects and tints seen in natural teeth, the user is able to reproduce natural shade effects and anomalies reliably and easily to create porcelain restorations with optimum shades and without having to produce complicated mixtures. AV.ZI.TI. Make Up Instant stains can be used for the Creation AV, Creation ZI and Creation TI techniques.

PROPERTIES

Creation AV.ZI.TI. Make Up Instant stains are fine grained, fluorescent, can be mixed together and reproduce the tooth shade effectively and harmoniously. The shade effect can already be discerned subtly while the stain is being applied or mixed into the porcelain.

Creation AV.ZI.TI. Make Up Instant Glaze provides the restoration with a silky sheen. Mix a small amount of glaze with the fluid and apply a thin coat to the restoration. Once the glaze has been applied, the shade can still be adjusted with Creation AV.ZI.TI. Make Up Instant. Lower the firing platform and allow the restoration to dry until the surface turns white.

Creation Willi Geller International AG, Oberneuhofstrasse 5, 6340 Baar, Switzerland
Fon: +41 41.768 90 19, Fax: +41 41.760 45 10, info@creation-willigeller.com, www.creation-willigeller.com

Technical inquiries: technic@creation-willigeller.com
Errors and printing errors excepted. Subject to change at any time.

Made by Klema Dentalprodukte GmbH, Koblacherstr. 3a, 6812 Meiningen, Austria

