

CC MAKE UP



COLOUR CHART

GLAZE MU/L	MU/L-71 white	MU/L-72 yellow	MU/L-73 orange	MU/L-74 ochre	MU/L-75 medium brown	MU/L-76 red brown	MU/L-77 khaki
	MU/L-78 olive	MU/L-79 grey	MU/L-80 pink	MU/L-81 lilac	MU/L-82 blue	MU/L-83 black	GL glaze

These stains have a fine grain structure and are transparent. They should be applied with a thin wash like watercolours, for internal and external staining.

MARKER MU/M	MU/M-91 red	MU/M-92 yellow	MU/M-93 honey	MU/M-94 brown	MU/M-95 dark brown	MU/M-96 brown grey
	CR-L CRACK LINER			ILLUSION		

These stains have a larger grain structure and the colour is also very intense. They can be used for example to simulate tartar or plaque deposits. They should not be used near the gingival margin, nor should they be used internally.

Crack Liner

To create fine cracks

Illusion

To create illusion of depth and to lower the value

CC FIRING CHART

	Preheating Temperature	Drying Time	Raise of Temperature	V	Final Temperature	Holding Time	Appearance
Glaze Firing	600 °C	2 min.	55 °C/min.	-	930 °C	-	Shining
Glaze Firing with Glaze Powder	600 °C	2 min.	55 °C/min.	-	900 °C	1 min.	

AV FIRING CHART

	Preheating Temperature	Drying Time	Raise of Temperature	V	Final Temperature	Holding Time	Appearance
Glaze Firing	600 °C	2 min.	55 °C/min.	-	910 °C	-	Shining
Glaze Firing with Glaze Powder	480 °C	2 min.	45 °C/min.	-	880 °C	1 min.	Shining

These firing parameters represent standard values and have to be adjusted to respective situation. Decisive is the firing result.

Creation Willi Geller International AG, Oberneuhofstrasse 5, 6340 Baar, Switzerland
 Fon: +41 41.768 90 19, Fax: +41 41.760 45 10, info@creation-willigeller.com, www.creation-willigeller.com

Technical inquiries: technic@creation-willigeller.com
 Errors and printing errors excepted. Subject to change at any time.

Made by Klema Dentalprodukte GmbH, Koblacherstr. 3a, 6812 Meiningen, Austria