



SET UP TEMPLATES INSTRUCTIONS FOR USE

The curved set-up templates are used for setting up full dentures according to the occlusion curves of Spee and Wilson. Once they are on the template holder, the templates are adjustable in inclination, and in the horizontal and vertical planes. The flat template can be used as an orientation guide for average-value mounting of the mandible. The part number for the Splitex (magnetic) version of the template holder is 46260.

Template Radius	Part #	Cusp Degrees	Type of Tooth	Manufacturer
100 mm	47741	0	WeroLux, Polystar	Werchan
			Biodent, Articon,	De Trey
125 mm	47742	15-16	Orthognat, n, t, k, Odilux	Lindau Teeth
				Ivoclar, Weithaas
140 mm	47744	25-28	Cuspiform	Vita
160 mm	47746	30-35	Othocal	Lindau Teeth
Flat	47740			

1. Attach the magnetic template holder to the magnet on the upper arm of the articulator.
2. Choose the appropriate template by radius desired, or choose the flat template for average-value mounting.
3. Loosen the screw on the template holder, slide the template on, and tighten the screw.
4. Loosen the T-clamp and align the curved template on the lower bite block.
5. Tighten the T-clamp and set up the teeth.