



Imagine™ Full Contour Zirconia

Thank you for purchasing Imagine™ full contour zirconia from Jensen Dental. Imagine™ is as translucent as pressed all ceramic and mimics the esthetics of natural teeth. With twice the strength of lithium disilicate (769 mPa), you can produce bridges up to three units in length with this new material.

IMPORTANT - PLEASE READ BEFORE USE

To ensure the best possible results Preciso Clients should confirm they have the necessary CAD and CAM configuration files before you begin working with Imagine. If you are unsure please contact Jensen Technical.

Lava Form, CNC500 & M500 clients must have T1, T2 and T3 burs loaded in their mill before using Imagine 18mm frame material.

IMAGINE SHADING TIPS

The current shading liquids on the market will work with Imagine but almost always result in shades that are about ½ a shade too light in chroma. A small amount of stain is recommended for dialing in the desired shade during the stain and glaze process.

Using Lava Plus Liquids

- Use Lava Plus liquids at 100% strength
- Shades tend to be slightly lighter than desired so 7 coats of dentin are recommended
- When using the dipping technique, use the liquid at 100% strength and let stand in the liquid for 2 minutes.

Using Tanaka Liquids

- Use Tanaka liquids at 100% strength
- Use 4-5 coats of dentin depending on the level of chroma desired

JENSEN SUPPORT

Jensen Technical: 800-528-5531
Jensen Sales: 800-243-2000

